# Manual de Instruções – Espaçador AgaChamber® Extra / User’s Guide – AgaChamber® Spacer Extra / Manual de Instrucciones – Espaciador AgaChamber® Extra

## PORTUGUÊS

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## ENGLISH

### Instructions for Use - AgaChamber® Spacer Extra - HOME / HOSP models

#### EXTERNAL USE / FOR HUMAN USE - ADULT AND CHILD

#### PRESENTATIONS:

Adult/Child HOME/HOSP Model	From age 3 on up
Baby HOME/HOSP Model	From 0 to 2 years of age
TRAC HOME / HOSP model	No age restriction

Package containing 01 Spacer with one mask.

Available colors: Yellow, Blue, Orange, Purple, Pink, Green and Red.

**Learn about the components of the AgaChamber® Spacer Extra Image 1 \*found at the end of the Instruction Sheet.**

**Notes:** TRAC HOME / HOSP model is formed only by parts 1, 2, 3 and 4, as in tracheostomized patients, medication is administered directly into the trachea.

#### COMPOSITION

Polypropylene, Pigments, Elastomers and Silicone.

#### USER INFORMATION:

##### WHAT IS THE AgaChamber® Spacer Extra - HOME / HOSP MODEL?

It is a health care product, intended for home and hospital use, it is reusable, provided the disinfection or sterilization is performed.

##### WHAT IS THIS PRODUCT FOR?

To assist and optimize the administration of inhaled aerosol medications, available in pressurized dispensers, popularly called "inhaler".

##### HOW DOES THIS PRODUCT WORK?

It works by directing the jet of the spray and allows the retention of large particles (propellant) in order to avoid them being deposited in the user's mouth or oropharynx, while simultaneously allowing the smaller particles (active substance) to reach the bronchi and small airways.

It has a valve (part nº3) of great importance to ensure the spacer's effectiveness, as it ensures a unidirectional flow of the medication (for inspiration), preventing the exhaled air to go into the chamber (barrier against return). This way, it avoids the dilution of the medication during the next spray, ensuring delivery of the complete dose.

The product was developed to be used with mask, according to each patient - children, adults or the elderly - with medical orientation.

The mask creates a closed environment, so that the user that has difficulty synchronizing the timing of their inspiration with the activation of the spray, receives the full dose of the medication, as it will be retained in the mask until the user inhales. The small valve of the mask is fixed and should not be removed under any circumstances even for cleaning, because it's function is to allow the air outlet patient's during exhalation and block the entry of air from the exterior.

##### FOR WHAT / FOR WHOM IS THIS PRODUCT INDICATED?

Indicated for patients with respiratory impairment (asthma, bronchitis, COPD, etc.) who are undergoing treatment or being accompanied by a healthcare professional.

##### WHEN SHOULD I NOT USE THIS PRODUCT? / WARNINGS

Keep AgaChamber® Spacer Extra out of children's reach, as it is a healthcare product-and not a toy.

When using on children, adult supervision is indispensable.

The product should be used only for the purpose in which it is intended for. The small valve in the mask (part nº 7) should never be removed, even during cleaning.

THE LARGE VALVE (PART Nº 3) IS ESSENTIAL FOR THE ADMINISTRATION OF THE MEDICATION TO ENSURE THE SAFETY AND FUNCTIONALITY OF THE PRODUCT, THEREFORE DO NOT REMOVE IT. IT SHOULD ONLY BE REMOVED DURING THE CLEANING PROCESS.

##### PRECAUTIONS

Proceed with the cleaning of the product before the first time using and after each use, following the recommendations described in this Instruction sheet.

**Attention:** At the discretion of the physician or healthcare provider, depending on as the criticality of the patient's clinical state, the disinfection or sterilization of the product before first use is recommended.

Before using the AgaChamber® Spacer Extra, ensure that there are no foreign objects inside the chamber. If so, repeat the cleaning procedure.

The assembly of the product outside the terms and orientations established on this instruction sheet can cause leakage of the medication and consequentially reduce the effectiveness for the use in which it is was intended for.

##### HOW TO SANITIZE THIS PRODUCT?

Disassemble the AgaChamber® Spacer Extra without removing the valve in mask (it should always stay fixed). Wash all parts under cold running water, using a few drops of neutral liquid detergent, a non-abrasive soft sponge and water. Rinse thoroughly.

Let the excess water drain out. Do not rub to dry. Allow it to dry natural, with the chamber (part nº 2) in the vertical position.

##### HOW TO DISINFECT / STERILIZE THIS PRODUCT?

So that the product may be used in a hospital environment, it is recommended that in addition to cleaning, a method of disinfection and or sterilization should be adopted after each patient according to the preference of the medical professional responsible.

Two methods were tested:

##### FOR DISINFECTION

Immersion in sodium hypochlorite (in concentrations of 0.02 and 0.5%) for 30 minutes.

##### FOR STERILIZATION

Submission to moist heat under pressure (autoclaving) at 121 °C for 20 minutes.

**Note:** The product can be reused for up to 10 disinfection/sterilization cycles. Opacity may occur in the chamber of the device, after the first cycle, however this does not compromise its functionality and effectiveness.

If the physician decides to provide the AgaChamber® Extra Spacer for the patient to take home, it is recommended that they provide instructions on how to use the product.

##### HOW TO ASSEMBLE THE AgaChamber® Spacer Extra?

The product should only be assembled when all parts are completely dry.

Assemble the product in the order shown in **Image 1\*** at the end of this instruction page.

##### STEP BY STEP:

Attach the base (1) in the chamber (2);

With the chamber vertically, position the side holes of the valve (3) in the direction of the fixing pins of the chamber and attach them perfectly. Both sides of the valve are identical;

After fixing the valve (3), connect the nozzle (4) to the same end of the chamber where the valve was fixed, screwing it to the end so that it is firm. Then firmly fit the mask (6) to the device;

If you will not use the Spacer immediately connect the nozzle cover (5), to protect it from dust and soiling.

##### HOW SHOULD I USE THIS PRODUCT?

After cleaning and/or sterilizing the product, make sure that the product is assembled with all its parts, as in **Image 1\*** illustrated at the end of this instruction page.

**Notes:** At the end of the application, to remove the spray (inhaler) from the base of the AgaChamber® Spacer Extra, secure the base with one hand and remove the inhaler with the other.

For doubts regarding the dosage and frequency of use of your medication, consult to your physician.

##### IMPORTANT INFORMATION

This product does not contain latex;

This product does not contain bisphenol A;

In case of doubts regarding the performance or use of the product, please preferably contact physician for appropriate instructions.

Undetermined validity, replacement suggested within 24 months after use.

##### CONDITIONS FOR STORAGE AND TRANSPORT

The product should be kept at room temperature, protected from dust and direct sunlight and sheltered from impacts.

To guarantee the product lifetime carefully follow all instructions in this manual.

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