# Manual de Instruções – Espaçador *AgaChamber*® VET / Manual de Instrucciones – Espaciador *AgaChamber*® VET / User’s Guide – *AgaChamber*® Spacer VET

## PORTUGUÊS

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EXTERNAL USE / VETERINARY USE

#### PRESENTATIONS

|                                                                      |
|----------------------------------------------------------------------|
| Models: Dogs and Cats                                                |
| Package containing: 01 Spacer with 02 masks in sizes (S/M) or (M/L). |
| Available color: Green.                                              |

#### INTENDED USE

It is an accessory for veterinary use, indicated to aid and optimize the administration of aerosol inhalation drugs, popularly called “puffers”, in domestic animals (dogs and cats) with respiratory deficiency (asthma, bronchitis, etc.) provided they are under treatment or care of a veterinarian.

#### HOW DOES THIS PRODUCT WORK?

The *AgaChamber*® VET Spacer acts by directing the drug jet and providing retention of large particles (propellant), preventing deposition in the animal's mouth or oropharynx, allowing smaller particles (active substances of the drug) to reach the small airways. The large valve (3) ensures the unidirectional flow of the drug, avoiding dilution of the drug, ensuring proper reception of the dosage.

#### BEFORE USING

The *AgaChamber*® VET Spacer chamber can be used at the time of purchase. Be sure to read the instructions for use and instructions prescribed by the veterinarian for use of the medicine and do not use the product if you notice any manufacturing defects. The chamber comes with two masks in S/M or M/L size. Select the mask that best fits the animal, covering the nose and mouth. The mask should not cover the eyes of the animal.

#### WARNINGS

- Keep the *AgaChamber*® VET Spacer out of reach of children and animals as it is a veterinary accessory containing small parts. It is not a toy.
- The product is for individual use.
- The product should be used only for its intended use.
- The product should not have the fixed nozzle valve (4) removed even during product sanitization.
- The product should have the large valve (3) removed for sanitization being careful not to lose it, as its use is essential to the effective use of the product.

#### PRECAUTIONS

- Perform product cleaning before the first use and after each use following the recommendations described in this Manual.
- Before using the *AgaChamber*® VET Spacer, make sure that there are no foreign objects inside the chamber. If so, remove and perform complete product hygiene.
- Be careful when handling the chamber (2), especially when sanitizing as it may crack or even break in the event of a strong impact.
- The assembly of the product outside the terms of this instruction for use can cause leakage of the medicine with consequent reduction of the effectiveness in the intended use.

#### IMPORTANT INFORMATION

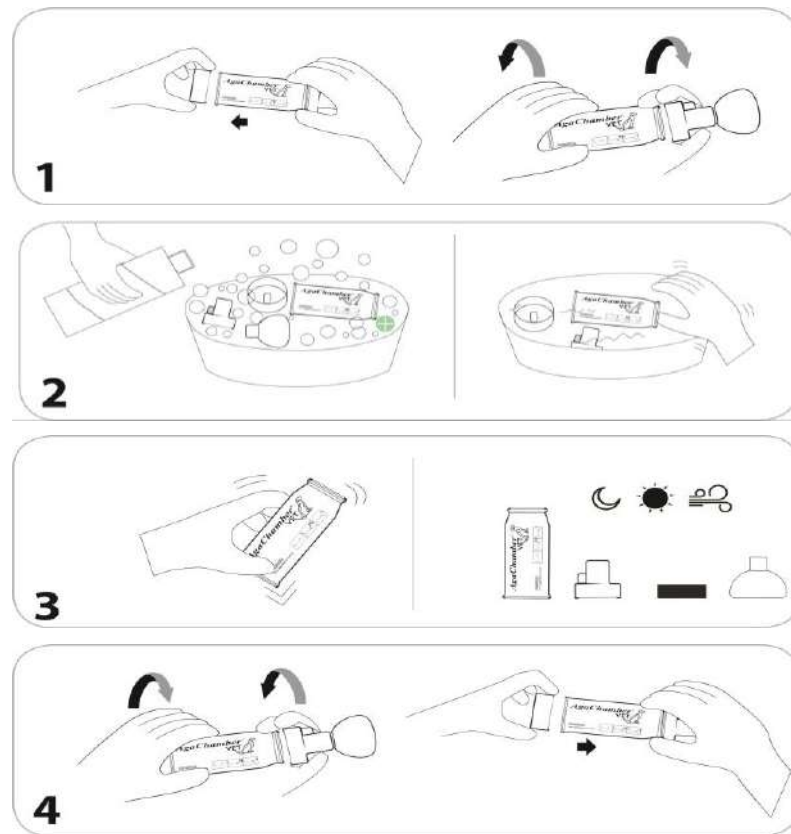
- This product does not contain latex or Bisphenol A;
- If you have questions about product performance or use, please contact your veterinarian for appropriate instructions.

#### STORAGE AND TRANSPORT CONDITIONS

The product should be kept at room temperature, protected from dust or direct sunlight and protected from impact.

# Manual de Instruções – Espaçador *AgaChamber*® VET / Manual de Instrucciones – Espaciador *AgaChamber*® VET / User's Guide – *AgaChamber*® Spacer VET

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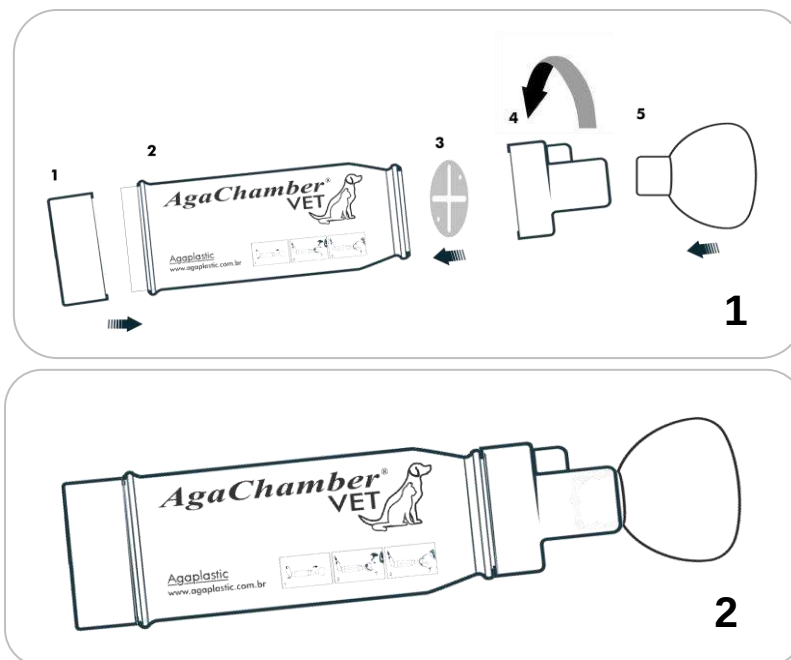
EN - Assemble the product in the order shown in the figure below:

- |                                                 |                                                                             |
|-------------------------------------------------|-----------------------------------------------------------------------------|
| 1 Base                                          | 4 Bocal com válvula fixa / Bocal con válvula fija / Nozzle with fixed valve |
| 2 Câmara / Cámara / Chamber                     | 5 Máscaras (P/M) ou (M/G) / Máscara (P/M) ou (M/G) / Mask (S/M) or (M/L)    |
| 3 Válvula grande / Válvula grande / Large valve |                                                                             |

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## ENGLISH

### HOW SHOULD I SANITIZE THIS PRODUCT?

Sanitize the product in the order shown in the illustration in Figure 1.

**1** Remove the *AgaChamber*® VET Spacer, without removing the smaller fixed valve from the nozzle (4). **2** Wash all parts in **cold running water** using a few drops of mild detergent, non-rough sponge and water. **3** Allow excess water to drain naturally to dry with chamber (2) in an upright position. **4** Assemble the product *AgaChamber*® VET Spacer.

**Note:** DO NOT USE HOT OR BOILING WATER.

### HOW SHOULD I ASSEMBLE THIS PRODUCT?

The product should only be assembled when all parts are completely clean and dry. Assemble the product in the order shown in the illustration in Figure 2.

**1** Attach the base (1) to the chamber (2). **2** With the chamber upright, position the large valve side holes (3) in the direction of the chamber retaining pins and snap them into place. Both sides of the valve are identical; **3** After fixing, fit the nozzle (4) on the same end of the chamber where the valve was fixed; **4** Fit the mask (5) directly into the nozzle (4).

### HOW SHOULD I USE THIS PRODUCT?

After sanitizing the product, make sure it is assembled with all parts as shown in the second image in Figure 2. Use this product as shown in Figure 3.

**1** Remove the protective spray cap from the medicine (puffer) prescribed by the vet. Shake it well to ensure proper release of medication. Insert the medicine spray (puffer) into the base of the *Agachamber*® VET Spacer. Make sure it is firmly seated. Fit the mask in the nozzle and apply it to the animal's face. **2** Keep the mask tightly sealed to the animal's face for as long as the veterinarian recommends. Make sure that the mouth and nose are well covered by the mask.

**Note:** If a second or more doses are required, follow the pause time (between doses) recommended by the veterinarian or as per the package leaflet.

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Company with Quality Management System

ISO 13485:2016

Rev.01-08/2019