Porta-Limas

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Endo Files Case

USE INSTRUCTIONS

CASE FOR ENDODONTIC FILES: The Indusbello file case allows the accommodation of endodontic files for sterilization, storage and transport. The file holder has nine compartments to accommodate endodontic files in a practical and safe way. The Indusbello file case has a transparent lid that allows viewing of the internal contents.

COMPOSITION: Thermoplastic resin. Note: Accessories such as profiles, supports and blankets are made of silicone.

INDICATION: Accommodate endodontic files for sterilization, storage and/or transport.

INSTRUCTIONS FOR USE: Open the case by sliding the cover upwards. Place one file in each compartment. Slide the lid down to close the case (see infographic). Pack the file holder in suitable material and carry out the sterilization process in an autoclave with a temperature of up to 135°C.

CLEANING: Clean manually with a soft sponge and neutral and/or enzymatic detergent. Wash with plenty of water.

DISINFECTION: Keep the case submerged in a 0.2% peracetic acid solution for ten minutes. Dry with a clean, dry cloth. Cleaning and disinfection must be carried out before each use.

WARNING: Respect the file capacity of the case. Place a file in each compartment to avoid damage to the material. Use the case in an autoclave with a temperature of up to 135°.

CAUTION: Do not touch the product to the inner walls of the autoclave. Avoid contact with chemicals (solvent and chlorinated hydrocarbons) as parts may deteriorate.

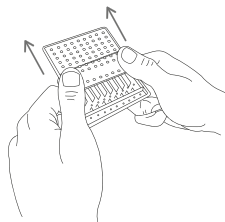
PACKAGING: Pack the product in an exclusive place, in closed cabinets, protected from any dirt.

SHELF LIFE OF THE PRODUCT: 05 years from the date of manufacture, in unbroken packaging. The shelf life of this product depends on the care taken during handling.

DISPOSAL: This product must be disposed of in accordance with current local environmental legislation.


1

Open the case by sliding the cover upwards. Place one file in each compartment.

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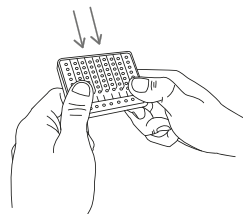Medical device (Europe).

European Authorized Representative.

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