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Mascarilla desechable
Disposable mask

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INFORMACIÓN TÉCNICA
TECHNICAL INFORMATION

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INFORMAÇÕES

PRODUTO

2i máscara descartável

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White with elastic: Pack of 10 or 50 units

FEATURES

2i masks are made with a triple layer of TNT (Non-Woven Fabric), a breathable and soft-touch material. They do not contain latex or fiberglass, reducing the potential for allergic reactions. They are equipped with a nose clip that ensures a perfect fit to the face contour.

USAGE INDICATIONS

2i masks are especially indicated to protect against contact with particles and foreign bodies. They are widely used in a variety of sectors, such as chemical, pharmaceutical, and food industries, as well as in nutrition services.

SPECIFICATIONS AND COMPOSITION

Specifications:

Length: 40 cm

Color: White/Colored

Composition: 100% TNT (Non-Woven Fabric) with elastic.

STORAGE AND TRANSPORT

STORAGE AND WARNINGS

Non-sterile product.

Single-use product.

Should not be reused or reprocessed.

Should be discarded after use.

TRANSPORT - RISK CLASSIFICATION

This product is not classified as dangerous according to ANTT resolution no. 420 of February 12, 2004.

USAGE INSTRUCTIONS

Remove the mask from the package, holding it by the edges, avoiding touching the inside.

Position the mask in front of the face, covering the nose and mouth.

Adjust the nose clip, pressing it gently against the nose so that the mask fits comfortably.

Secure the elastic straps behind the ears, ensuring that the mask is secure and completely covering the nose and mouth.

APPLIED REGULATION

ANVISA Registration: 81130710009