# 09938 Unobject - Instrucao de Uso

## Storage and Conservation:

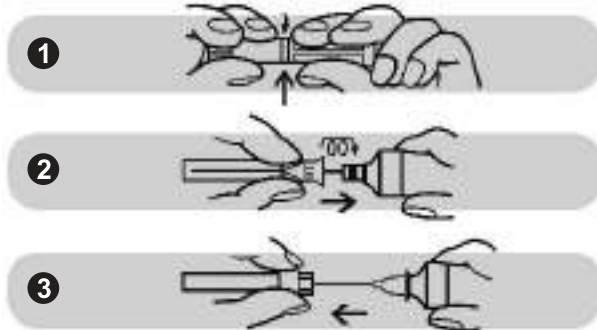
Keep the product in its original well closed package and store in a dry place. Shelf-life: 5 years.

## Warnings and Precautions:

- Do not use if package is damaged, open or wet.
- Do not use if seal is broken.
- During use, do not change needle direction while it is inside tissue.
- Never force the needle if there is any resistance.
- The needles should be disposed and destroyed immediately after use to avoid injuries or reuse by non-authorized people. Do not dispose without first isolate the needle with the cartridge case. Dispose preferably in special containers aimed for contaminated materials.
- Since they are used, needles should not be re-sterilized or reused.
- This product should be used by dentists only and just for the purposes described in these Instructions for Use.

## Instructions for Use:

Turn ends of the capsule until the rupture of the seal. Do not use if seal is broken. Remove the smaller cover. Thread the needle on the syringe and remove the protection.

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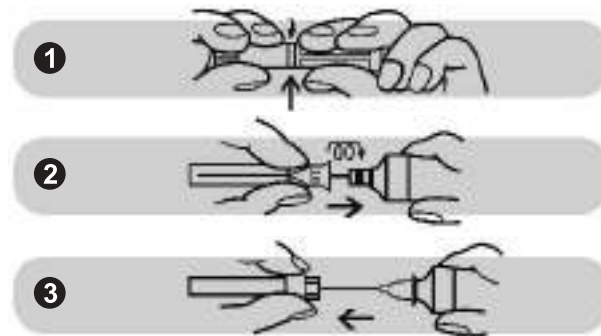
**UNOJECT**  
**Disposable gingival needle**  
*English*

**PROFESSIONAL USE ONLY**

Tri-beveled needles with thin walls which provide improved flow of the anesthetic solution. Sealed by melting weld and super safe protectors. Non pyrogenic. Sterilized by ethylene oxide.

**Composition:**

Made of special high technology stainless steel. Siliconized to soften the penetration of the cannula into tissues.

**Indications:**

For injection of anesthetics according to the techniques indicated for that size of needle.

**Presentations:**

- Box containing 100 sterilized disposable needles.  
 Size: 25mm x 0.3mm (30G) Short.
- Box containing 100 sterilized disposable needles.  
 Size: 16mm x 0.3mm (30G) X Short.
- Box containing 100 sterilized disposable needles .  
 Size: 38mm x 0.4mm (27G) Long.

**STERILE NEEDLE OF SINGLE USE.**  
**THE SEAL GUARANTEES THE STERILITY OF THE NEEDLE.**  
**THE NEEDLES SHOULD BE DISPOSED AFTER USE.**  
**RECYCLING IS FORBIDDEN.**